

SDM amalgam Trap Preventive Maintenance

<i>Room</i>	<i>Unit</i>	<i>Assigned to</i>	<i>Assign Date</i>	<i>Due Date</i>	<i>Notes</i>
702	A				
702	B				
702	C				
702	D				
702	E				
702	F				
704	A				
704	B				
704	C				
704	D				
704	5				
704	6				
710	A				
710	B				
710	C				
710	D				
710	E				
710	F				
712	A				
712	B				
712	C				
712	D				
712	E				
712	F				

- Don full PPE (Gloves, safety glasses, and mask)
- Flush evacuation system with approved solution
- Flush evacuator system again if infectious materials are present
- Remove amalgam trap
- Remove excessive residue from trap walls using cotton-tipped applicator or spray from a Air/Water syringe tip
- Dispose of contaminated cleaning supplies should be in accordance of our Infection Control guidelines
- Replace amalgam trap
- Replace outlet port cap
- Clean work area in accordance of our infection control guideline
- Remove PPE to wash and dry hands

Traps should be checked and logged in binder stored in Dispensary weekly in accordance with our Infection Control Procedure